Instructions for use

myQuickmat Forte kit

sectional
matrix
systems

Index

EN - English	2	SE - Svenska	20
DE - Deutsch	4	DK - Dansk	22
FR - Français	6	PL - Polski	24
IT - Italiano	8	CZ - Čeština	26
ES - Español	10	TK - Türkçe	28
PT - Português	12	GR - ελληνικά	30
NL - Nederlands	14	JP - 日本語	32
NO - Norsk	16	CN - 中国	34
FI - Suomi	18	AR - العربية	36

Class II restoration using myQuickmat Forte kit

Sectional matrix system developed to offer **strong** teeth separation. MyRing Forte separates the teeth. Its anatomically shaped, reusable tines ensure perfect matrix adaptation and myWedge forms the matrix band to the cervical area of the tooth.


Instructions for use:


1. Prepare the cavity

Remove the carious lesion and prepare the tooth with a conservative Class II preparation. Even if the contact was not totally broken the matrix can be placed.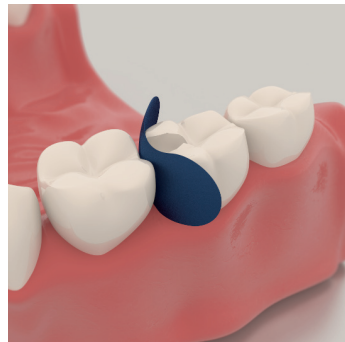


2. Place the matrix

Select an appropriate matrix band according to the tooth morphology and restoration type. Place the contoured sectional matrix with the concave edge towards the occlusal margin using your fingers or with tweezers.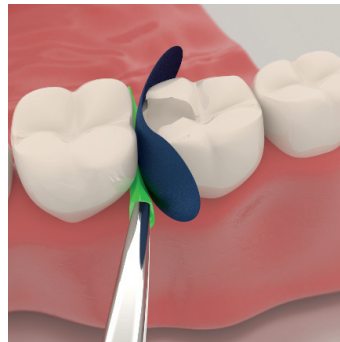


3. Wedging

Select the appropriate sized mywedge to close the gingival margin, and insert it firmly. When needed maintain a finger on the matrix band to avoid dislodging during wedging.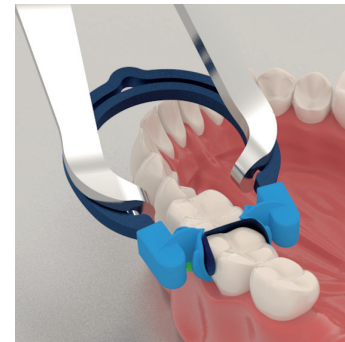


4. Place myRing Forte

Expand myRing Forte using Polydentia's myForceps; enlarge it just enough to position it on the tooth. Carefully place myRing Forte over the wedge as far gingivally as possible.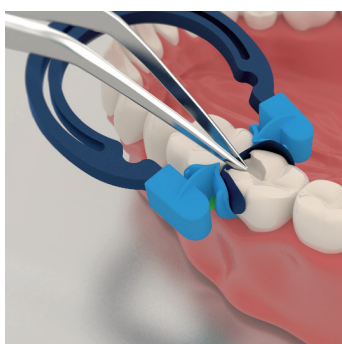


5. Burnish the matrix

For an excellent contact, burnish the band against the adjacent tooth using a ball burnisher or tweezer.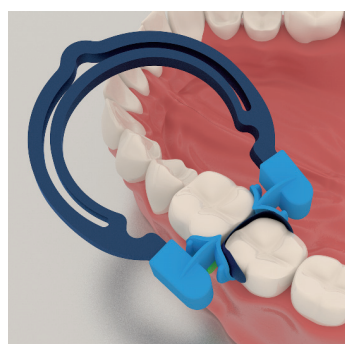


6. Restore the cavity

Restore the cavity according to the filling material manufacturer's instructions.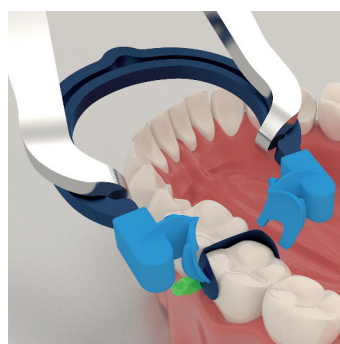

7. Remove the sectional matrix system

Remove myRing Forte, the wedge and the matrix. Our suggestion for an easy matrix removal is to use the special Polydentia myQuickmatrix Forceps.

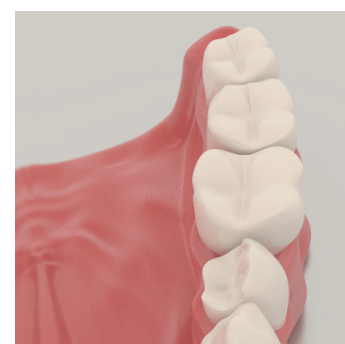8. Finishing and polishing

With thin burs remove any small excess of material then polish the restoration according to the finishing and polishing instruments manufacturer's recommendations.

Recommendations:

- 1 - Place a rubber dam
- 2 - Read Cleaning, Disinfection and Sterilization Instructions at www.polydentia.com

Tips & Tricks using myQuickmat Forte kit

Tines removal

Tines must be disassembled before cleaning and sterilizing. To remove the tines correctly, it is important to preserve the internal anatomical wings. Make sure to pull the tines by grasping the external part. For more information about cleaning and sterilizing refer to the document "cleaning and sterilization procedure"

Tines placement on myRing Forte

Tines are replaceable and supplied in pairs. You can easily detach the tines, avoid finger pressure on the internal anatomical wings to preserve them. Make sure tines are placed on myRingForte as shown in the picture below.

MOD and multiple tooth restorations







MyQuickmat Forte rings are designed to be stacked over one another to perform a MOD and multiple tooth restorations.

1. Prepare the cavities as described in the instructions for use, step 1.
2. Place matrices and wedges on both mesial and distal surfaces of the tooth being restored
3. Position myRing Forte
4. Complete the restoration as described in the instructions for use, steps 5 to 8.

Restoration of adjacent cavities

MyRing Forte tines are shaped to perfectly adapt the mesial and distal surfaces of two adjacent teeth. This feature allows the restoration of two adjacent cavities at the same time.


1. Prepare the cavities as described in the instructions for use, step 1.
2. Place matrices and wedges on both teeth being restored
3. Position myRing Forte
4. Complete your restoration as described in the instructions for use, steps 5 to 8.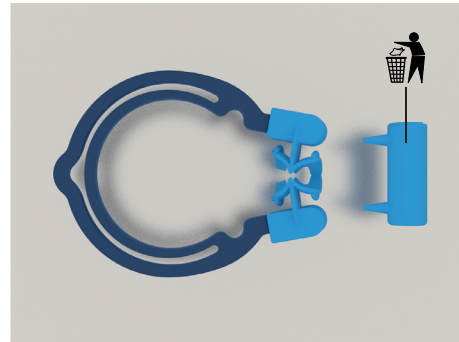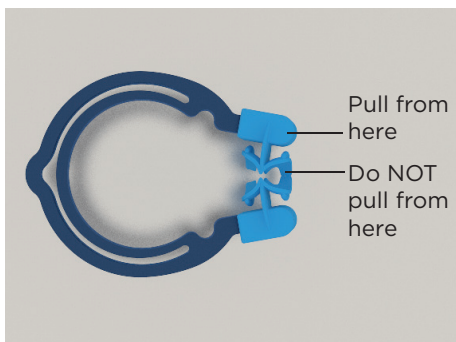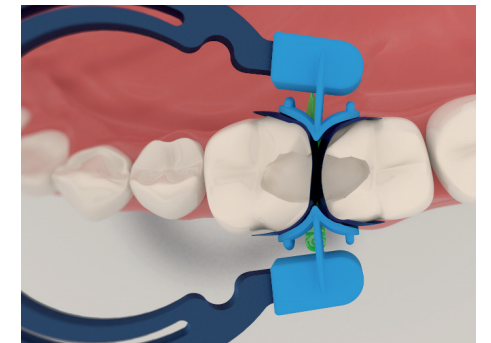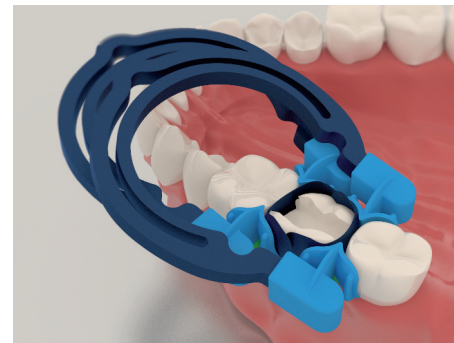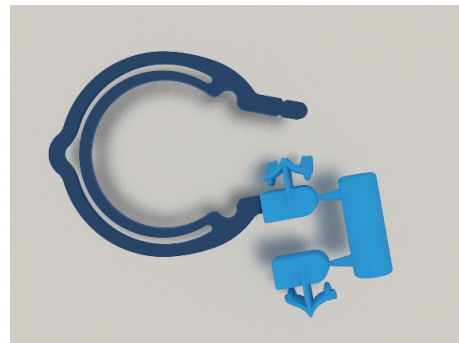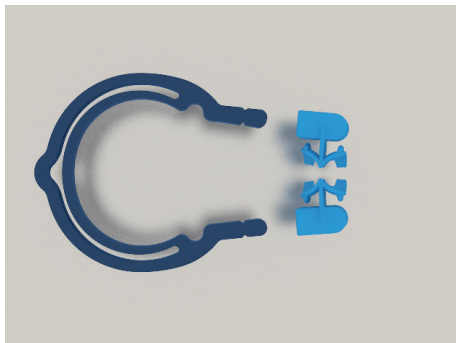

myRing Forte Reset

If myRing Forte is over-expanded it can be retensioned by using Polydentia's myForceps as follows:

- remove the tines first
- place myRing Forte within the notches on the forceps
- squeeze forceps closed to return myRing Forte in its perfect shape.

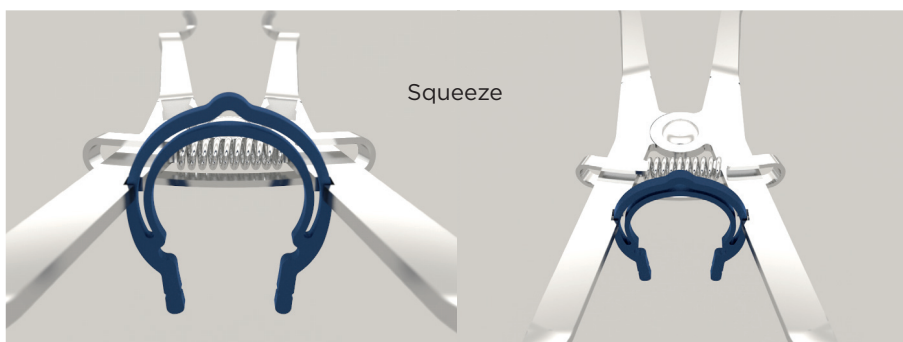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